ENGLISH

CONTENT

L-292136 Stainless Pyramid w/ o-ring (rated up to 100kg / 220lbs) (**fig. 3**).
L-692142 Pyramid Cap (**fig. 2**).
L-692141 Plastic washer M8 (**fig. 1b**)
L-692032 Pyramid Bolt w/ o-ring (**fig. 1**).
L-293049 Silicone Fill for 273 Pyramid (**fig. 4a**).

OR

L-292135 Titanium Pyramid w/spacer & o-ring (rated up to 166kg / 365lbs) (**fig. 3**).
L-692142 Pyramid Cap (**fig. 2**).
L-692141 Plastic washer M8 (**fig. 1b**)
L-292032 Pyramid Bolt long w/ o-ring (**fig. 1**).
L-293048 Silicone Fill for 272 Pyramid (**fig. 4a**).

USE WITH L-211000 ICELOCK CLUTCH 211

- Flatten distal end of Plaster Cast (8) to center Delrin Guide (7). Attach with nails provided or double-sided tape maintaining original pin alignment.
- Apply inner PVA bag and tie it around 'neck' on Delrin Guide.
- Trim bag around string and place Silicone Seal (6) over 'neck' of Delrin Guide (7), onto PVA bag. Bond Silicone Fill (4a) to Lock Body (4b) using double-sided tape.
- Place Lock Body (4b) and connected Silicone Fill (4a) onto Delrin Guide (7), making sure the Dummy Screw (5) for Clutch Mechanism (9) is correctly positioned
- Place Pyramid (3) onto Lock Body (4b), making sure that o-ring in groove on pyramid is in place.
- Put Pyramid Cap (2) and plastic washer (1b) onto Pyramid (3) and secure them with Pyramid Bolt (1) using 6mm hexagonal key (make sure that o-ring on Bolt is in place).
- Tighten the bolt (1) using 6mm hexagonal key

NOTE: Be sure to tighten (1-2Nm) so that resin will not leak in between components.
- Apply clay into hexagon hole (A) on bolt and groove (D) on Dummy Screw.
- Tie lay-up material around Lock Body (4b) and Silicone Fill (4a) and into groove (C) on Pyramid and groove (B) on Delrin Cap.

IMPORTANT: Tie at least two layers of fibers in groove (B) on Delrin Cap.
- Apply outer PVA bag and pour resin.
- When curing has finished, grind down to Pyramid Bolt (1) and Dummy Screw (5) and remove.
- Remove Pyramid Cap (2).

NOTE: DO NOT grind material away from distal end of socket (E) since that reduces strength of connection between socket and pyramid.
- Remove socket from plaster cast.
- Insert and tighten Clutch Mechanism (9) to 15Nm (11ft.lb) with 17mm torque wrench (VERY IMPORTANT).**

NOTE: Push Button (10) may be shortened by cutting it with hacksaw.

ATTACHMENT PIN

Apply Loctite 410 or 411 onto threads (F) on Attachment Pin (11) and screw it into Locking Liner. Torque Attachment Pin to 4Nm (3ft.lb). Put Protector Cap on Pin before donning liner to prevent damage. Remove Cap before donning socket. Also available:

L-292003 Attachment Pin clutch, short w/cap (7 notches).
L-292005 Attachment Pin clutch, long w/cap (17 notches).

Note: Use attachment pins for Icelock 200 series only.

T-wrench supplied is specially designed to wind Push Button (10) on Clutch Mechanism (9) when donning socket. Use of any other tool may result in over tightening.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If an-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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©Copyright Össur 2018