Instructions for Use

4-HOLE ADAPTER

EN		Instructions for Use	3
DE		Gebrauchsanweisung	5
FR		Notice d'utilisation	7
ES		Instrucciones para el uso	9
IT		Istruzioni per l'uso	11
NL		Gebruiksaanwijzing	13
PT		Instruções de Utilização	15
TR		Kullanım Talimatları	17

ÖSSUR 4-HOLE ADAPTER, PIVOTED

The Össur 4-hole adapters, which can be pivoted around 360°, serve to connect the socket, i.e. the container cup and the Össur modular system.

The fine-notched rotatable adjustment offers the option of setting the internal or external rotation quickly and simply after the prosthesis socket has been finished without having to take the unit apart.

Due to the special construction, the adapter ensures that the unit can still be turned through 360° around the threaded rod, even after being screwed tight.

Important: when choosing the adapter, please observe the approved maximum body weight. Add possible extra loads to the patient's body weight (e.g. bottle crates) to avoid any possibility of the maximum body weight being exceeded.

THE FOLLOWING POINTS MUST BE OBSERVED WHEN USING THE ÖSSUR 4-HOLE ADAPTER:

1. When mounting the adapter, make sure that the whole surfaces of both the base of the socket or the socket connector and the adapter are in contact.
2. The M6 hexagon socket screws must always be tightened to the specifications of the opposite adapter or counterpart (socket connector, socket adapter).
3. Applies only to the Össur 4-hole pyramid receivers: the M8 threaded rods of the adapter must always be tightened to a torque of 15 Nm and secured with Loctite.
4. **WARNING:** any manipulation whatsoever leads to premature wear-and-tear (fracture) of the adapter.

TECHNICAL DATA

Art. No.	Name	Own weight	Maximum body weight	Material
M.625.400	Össur 4-hole pyramid adapter, pivoted	approx. 145 g	136 kg	St
M.626.400	Össur 4-hole pyramid receiver, pivoted	approx. 115 g	136 kg	St

MAINTENANCE

The Össur Adapter must be inspected at least every 6 months. In the course of this inspection, check:

- the general state of the adapter
- for damage
- for wear-and-tear
- the suitability of the patient (e.g. max. body weight, mobility)
- torque
- if the user has fallen, the entire prosthesis system must be checked meticulously.

LIABILITY

In the event of a claim: Össur will only consider complaints submitted with a copy of the delivery note or invoice from Össur and exact details of the reason for returning the joint. A manufacturer can only be held liable for the failure of his own fittings. Any liability of the manufacturer in excess of this is only possible if his fittings are demonstrably the cause of the damage or loss of function of fittings from other manufacturers.

GUARANTEE

For Össur 4-hole adapter, there is a guarantee period of 24 months.

CE MARK, COMPATIBILITY

All parts have been tested to EC Directive 93/42/EEC and bear the CE mark.

GENERAL CONDITIONS:

This product is recommended for use by one patient only. Product liability pursuant to the Medical Devices Act becomes null and void if it is used for treating more than one patient.

The compatibility of Össur products with fittings from other manufacturers is guaranteed as long as the following points are satisfied:

- The fittings are only used in compliance with their intended purpose (see Instructions for Use).
- If fittings with different max. weight limits are used, the maximum loading corresponds to that of the weakest fitting.
- The use of tested individual components does not relieve the prosthetist from his duty to check by all possible means the combination of fittings for their suitability, correct assembly and safety.
- If there are any indications that a certain combination of fittings does not guarantee the required level of safety, then the fittings must not be combined.
- The prostheses must be built in accordance with the generally recognised professional rules of the orthopaedic technical trade.
- If the patient's situation changes (e.g. body weight, degree of activity, etc.) the prosthesis must be checked immediately.
- If the prosthesis has been subjected to disproportionately high stresses, e.g. during a fall, it must be inspected for any possible damage immediately by a specialist workshop.
- Safety-relevant regulations for individual fittings (e.g. maintenance intervals, see Instructions for Use) must be observed.

COMBINATIONS OF DIFFERENT FITTINGS:

The above general conditions apply, in addition:

- Use only fittings that comply with the requirements of ISO 10328.
- Use only fittings that comply with the requirements of the Medical Devices Act.
- The fittings must be designed for use with standard pyramid or 30 mm systems, and must be compatible with each other.

USEFUL LIFE

The maximum useful life of the Össur adapter is 36 months.

SAFETY WARNINGS

You must observe the alignment guidelines and instructions for use.

You must observe the maintenance and care guidelines.

This medical aid is for use by one patient only.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards is achieved only when Össur products are used with other recommended Össur components. If an unusual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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