ENGLISH

APPLICATION:

Proper application of the Stabilizer is important for immobilization and patient comfort. The two - piece design consists of a FRONT piece and BACK piece which are packaged as a set.

Always maintain the patient's head in neutral alignment. These directions are guidelines only; follow a qualified healthcare provider's instructions.

1. Insert the plastic reinforcement of the Collar FRONT piece into the top of the Stabilizer FRONT piece. Slide the pieces together until the holes are in alignment. Repeat for the Collar BACK piece and Stabilizer BACK piece.
2. After the Stabilizer and Collar holes are in alignment, insert the locking plugs. Detach the nylon fastener after the plugs are securely locked into position and the two pieces cannot be separated.
3. After properly applying the Collar, insert the hook and loop straps located on the Stabilizer BACK piece through the buckles located on the Stabilizer FRONT piece.
4. Pull both straps simultaneously and with equal pressure toward the Stabilizer BACK piece. Verify that all components are centered on the patient and the patient is in neutral alignment. Fasten strap securely and remove excess material, as necessary.

REMOVAL:

Before removing the Stabilizer and Collar, mark the position of the hook and loop strap location to ensure reapplication maintains the proper fit. With the Stabilizer and Collar secured on the patient, loosen and unbuckle the Stabilizer strap on the RIGHT side and then on the LEFT side. To remove the Collar, unfasten the hook and loop strap on the RIGHT side and then the LEFT side. Remove Stabilizer and Collar from the patient.

Plugs can be removed only when the Stabilizer is no longer necessary and only the Collar is required as instructed by a qualified healthcare professional.

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EN – Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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