ENGLISH

MEASUREMENT & SIZING

The Adult Philadelphia Halo will fit most average individuals when attached to the Philadelphia Collar.

The Pediatric Philadelphia Halo can be attached to both the Child and Infant Philadelphia Collar.

SIZE CHART & PRODUCT CODE

Size	Product code
Pediatric	OHP
Adult	OH

Custom sizes available upon request.

APPLICATION

Proper application of the Halo is important for immobilization and patient comfort. The two-piece design consists of a occipital support and a VelStretch™ strap which are packaged as a set.

1. Slide the plastic reinforcement of the Collar BACK piece through the top of the Halo occipital support slots. Slide the pieces together until they are in alignment.
2. After the Halo and Collar are in alignment, snap the pieces together until they are securely fastened.
3. After properly applying the Collar, wrap the Velstretch™ strap around the patient's head. Position the Velstretch™ material on the center of forehead.
4. Pull both straps simultaneously and with equal pressure toward the Halo occipital support. Slide straps through the buckels and fasten the straps. Verify that all components are centered on the patient and the patient is in neutral alignment.

REMOVAL

Before removing the Halo and Collar, mark the position of the Hook and loop strap location to ensure re-application maintains the proper fit.

With the Halo and Collar secured on the patient, loosen the Halo strap on the RIGHT side and then on the LEFT side. To remove the Collar, unfasten the Hook and loop strap on the RIGHT side and then the LEFT side. Remove Halo and Collar from the patient.

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EN – Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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