









Instructions for Use






---

OCCIAN<sup>®</sup> COLLAR BACK  
& OCCIAN<sup>®</sup> JR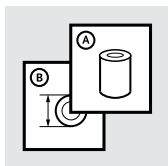

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	8
FR	Notice d'utilisation	12
ES	Instrucciones para el uso	16
IT	Istruzioni per l'uso	20
NO	Bruksanvisning	24
DA	Brugsanvisning	27
SV	Bruksanvisning	31
FI	Käyttöohjeet	35
NL	Gebruiksaanwijzing	39
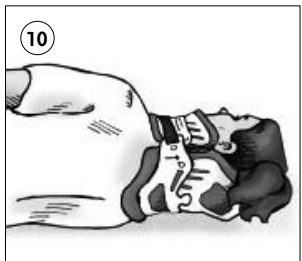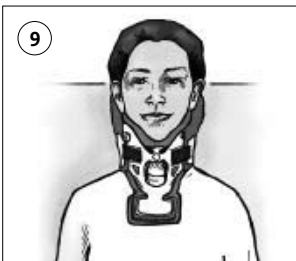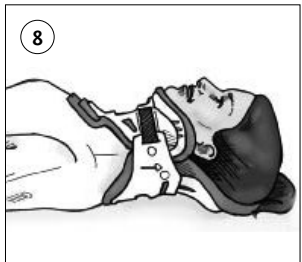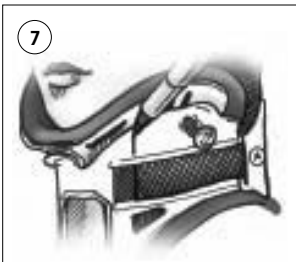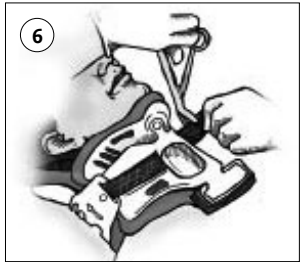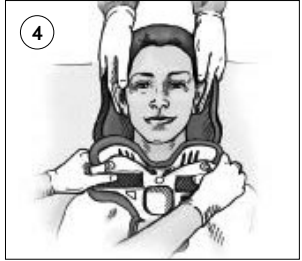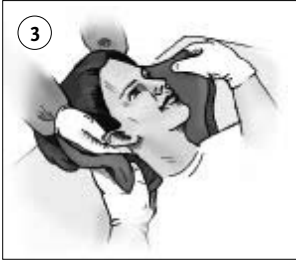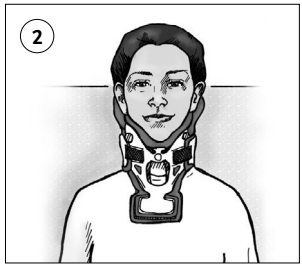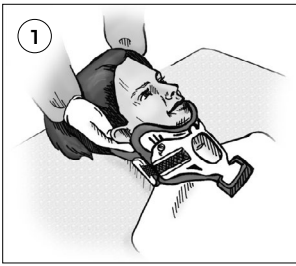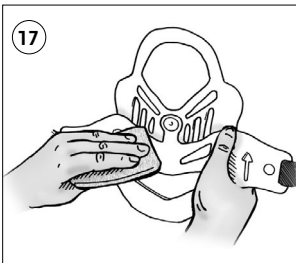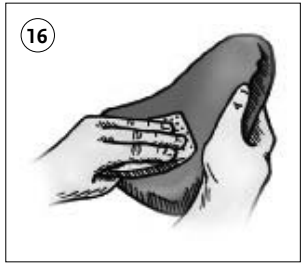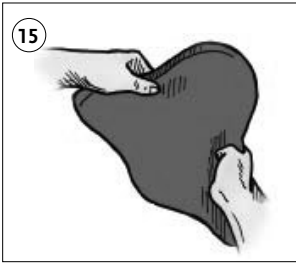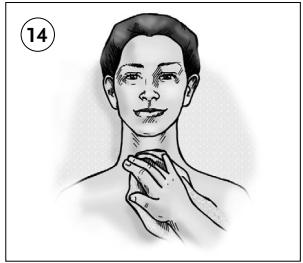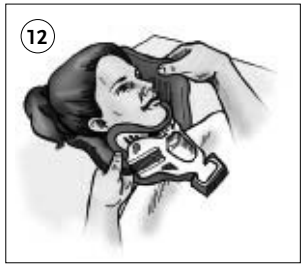
# ENGLISH

---

The Occian Collar Back / Occian Jr. is an accessory back piece indicated for patients with C-spine precautions who are at high risk of developing occipital breakdown or those desiring nighttime post-operative comfort. **The Occian Collar Back / Occian Jr. should be replaced with the appropriate size Miami J® Back when the patient is ambulatory.**

**CAUTION:** Federal law restricts this device to sale by, or on the order of, a physician.

Not made with natural rubber latex.

X-ray and CT lucent. MR safe.

Featuring IntuiTech™ advanced pressure management media.

## **APPROPRIATE USE OF THE OCCIAN COLLAR BACK**

The Occian Collar Back / Occian Jr. is an accessory back piece designed to replace the “ambulatory” collar back. This special back piece is made with advanced, pressure-relieving IntuiTech™ viscoelastic material. The Occian Collar Back / Occian Jr. lines the patient interface of the collar with a therapeutic surface to inhibit occipital ulcer formation. It also provides enhanced comfort during sleep when used with any collar.

## **HIGH RISK PATIENTS**

The Occian Collar Back / Occian Jr. is indicated for patients with c-spine precautions who are at high risk of developing occipital breakdown including:

- Candidates for specialty beds/surfaces.
- Patients who remain in a supine position for long periods of time related to respiratory, neurological or hemodynamic instability, pelvic fractures or skeletal traction.
- Patients who are not awake and alert within 24-48 hours after admission.
- Patients with poor skin assessment or pre-existing skin injuries, edema or blistering.
- Burn patients or diabetic patients who must wear a collar.
- Elective surgery patients desiring nighttime postoperative comfort.

Once a high-risk patient has been identified, initiate use of the Occian Collar Back / Occian Jr.

## **SIZE IT UP!**

The Occian Collar Back is universally sized to fit all adult Miami J® Fronts as well as the Miami Jr.® P3 Front. The Occian Jr.® Back is designed to fit the Miami Jr.® P2 Front. Refer to the Miami J® Sizing & Application Instructions to select the appropriate size Miami J® Collars.

## **SCOOP IT UP!**

### **APPLICATION INSTRUCTIONS**

#### **USE PROPER C-SPINE PRECAUTIONS:**

At least two persons are needed for initial collar placement: one to maintain the patient's head and neck in proper alignment, the second to fit the collar.

1. Remove any pillows from behind the patient's head. Position patient with arms to the side, shoulders down and head centrally aligned (**Figures 1 and 2**).
2. Remove the original collar back piece and slide the Occian Collar Back / Occian Jr. behind the patient's neck and center it (**Figure 3**). **Before applying Occian Collar Back / Occian Jr., ensure the IntuiTech™ padding extends beyond all plastic edges of the collar. Retain the original collar back for use when the patient becomes ambulatory.**
3. Flare the sides of the Miami J® Front out, slide it up the chest wall and **scoop it up** under the chin. Sides of the collar Front should be oriented up, off the trapezius, and toward the ears. **Place the plastic of the collar Front in between the Occian collar pad and the Occian collar plastic; this will ensure that the Occian pad is against the skin (Figure 4).**
4. While holding the Front securely, curl the ends snugly against the patient's neck. Apply the hook and loop strap and secure the opposite side of the collar in the same fashion. Tighten straps alternately, one at a time, to an equal length on both sides (**Figure 5**).
5. Hook and loop straps must be aligned symmetrically and oriented "blue-on-blue" to the Front hook sections. There should be equal amounts of excess hook and loop overhanging the Front blue hook sections. Excess hook and loop may be trimmed if desired (**Figure 6**).

## **COLLAR REMOVAL & SKIN CARE**

Keeping the Miami J® Collar with the Occian Collar Back / Occian Jr. and the skin beneath clean is an important part of treatment. Daily cleaning will help to prevent skin irritation. Inspect and assess the skin beneath the collar for breakdown once per shift. Clean the collar once per day or more often as needed for patients with excessive oral drainage or diaphoresis resulting in dampness of the padding.

Unless otherwise specified by the physician: **Do not remove the collar except to wash under it and clean the pads.**

Before removing the collar, note where the ends of the hook and loop straps are. When the collar is replaced, the straps should be in the same position. You may use a pen to mark the positions on the white plastic shell (**Figure 7**).

1. Position patient flat in bed without a pillow. Keep head in a neutral position. **Do not allow the neck to bend forward, backward or from side to side.** Position patient with arms to the side, shoulders down and head aligned centrally (**Figures 8 and 9**).
2. With physician approval only: Gently roll the patient onto his/her side. (Place a small pillow or folded towel under his/her cheek to prevent the head from tilting sideways) (**Figure 10**).
3. Undo the hook and loop strap on the top side of the collar and fold the Occian Collar Back / Occian Jr. down to clean and check the back of the patient's head for redness or irritation. Visualize and feel skin, scalp and under hair for any irritation or breakdown. **Make sure Occian Collar Back / Occian Jr. is replaced and securely re-fastened after skin check (Figure 11).**
4. Refasten the collar and return the patient to a supine position. **Wearing a beard may cause skin irritation in men. Redness on chin may indicate improper use of a pillow. If permitted, pillows must always be placed behind the head and shoulders.**
5. If unable to log-roll, undo the hook and loop straps. Remove the

Occian Collar Back / Occian Jr., sliding out from under neck if necessary (**Figure 12**).

The Occian Collar Back / Occian Jr. may be easily cleaned now. Patients in a Rotorest® bed may have the posterior neck accessed by opening the top hatch from underneath while a nurse holds the patient's head in neutral alignment from above.

6. While maintaining c-spine precautions, remove the Miami J® Front. Clean the collar Front following the instructions in the Miami J® Patient Handbook (**Figure 13**).
7. Palpate the back of the head and wash front and back of neck with mild facial soap and water and completely dry the skin. Observe for any redness or irritation under the collar, especially over bony areas like the chin, or clavicles(**Figure 14**).

## CLEANING THE OCCIAN BACK

Instructions below are for care of the Occian Collar Back / Occian Jr. only.

1. Peel the soiled Occian pad off (**Figure 15**).
2. Surface clean the pads only with mild facial soap and water or a non-chlorine hospital-grade disinfectant. **DO NOT use bleach or harsh detergents** (**Figure 16**).

3. Wipe the white plastic collar shell clean with mild facial soap and water (**Figure 17**).
4. Dry the collar and the surface of the pad thoroughly with a towel and re-attach the IntuiTech™ pad. Fold the pads in half with the logo-printed side next to the hook and loop; center the pad in the white shell (**Figure 18**).

Adjust pads as needed to make sure no plastic touches the skin.

## REPLACE COLLAR

1. Slide the Occian Collar Back / Occian Jr. carefully behind the patient's neck. Make sure it is centered evenly. **Long hair should be placed outside the collar** (**Figure 3**).
2. Flare the sides of the Front out, and scoop it up! Under the chin. Holding the Front securely, curl the ends snugly against the patient's neck. Fasten the hook and loop strap, returning to the marked end on collar, then secure the other side in the same way (**Figure 4**).

## DOCUMENTATION

Assess the collar for proper fit and patient comfort.

Documentation of assessment and inspection findings is to be noted in the nurses notes and includes the following:

- A. Skin assessment (notify physician if breakdown observed)
- B. Interventions initiated and patient response
- C. Collar care
- D. Post-care neurological exam
- E. Patient and caregiver instruction and level of understanding
- F. Any change in findings

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