

Instructions for Use

C180 ROCKET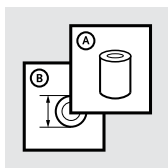

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	10
ES	Instrucciones para el uso	13
IT	Istruzioni per l'uso	16
NO	Bruksanvisning	20
DA	Brugsanvisning	23
SV	Bruksanvisning	26
FI	Käyttöohjeet	29
NL	Gebruiksaanwijzing	32
PT	Instruções de Utilização	35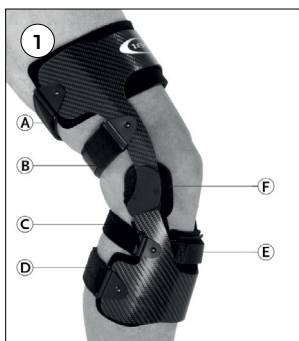

ENGLISH

LIST OF PARTS: (Figure 1)

- A. Top Strap
- B. Upper Cruciate Strap
- C. Lower Cruciate Strap
- D. Bottom Strap
- E. Anterior Tibial Strap
- F. Condyle Pad

INDICATIONS FOR USE

ACL, MCL, LCL, PCL, rotary and combined instabilities.

WARNING

This Knee Brace is a supportive brace only and is not intended or guaranteed to prevent knee injury. Purchaser agrees to the following:

1. Ossur accepts no responsibility for any injury which might occur while using Ossur knee bracing products.
2. With use of this type of device, it is possible to receive various types of skin irritation, including rashes and pressure points. If you experience any unusual reactions while using this product, consult your health care provider immediately.
3. If you have any questions as to how this knee brace applies to your own specific knee condition, consult your health care provider. If you do not agree to the above conditions, contact your brace provider before using this product.

BRACE APPLICATION

1. Unfasten all straps, including tibial adjustment strap, from the anterior side of the brace.
2. With the knee bent in approximately 90 degrees, place the brace over your knee such that the patella is centered between the condyle pads (Figure 2).

NOTE : This places the hinge slightly above the joint line so the brace can properly seat itself when you stand (Figure 3).

3. Beginning with the strap just below the knee (lower cruciate strap), fasten the strap snugly above the calf muscle as this will aid in the brace suspension (Figure 4).
4. Next, snugly fasten the bottom strap by inserting it through the D-ring.
5. Continue with the strap just above the knee (upper cruciate strap), followed by the top strap. Snugly fasten both straps (Figure 5).
6. Tibial strap should be secured last. Tighten strap so the brace is snug against the leg (Figure 6).

NOTE : Over-tightening the two top straps can cause the brace to migrate.

WARNING: If brace causes discomfort or irritation, discontinue use immediately and contact your health care provider.

BRACE MAINTENANCE

The C180 brace is designed for minimum maintenance and care. Follow these easy maintenance steps to keep your brace in good condition.

Cleaning

- Liners, straps and pads may be removed for cleaning.
- All liners may be hand washed using a sponge or soft cloth and 86F (30 °C) warm water with mild detergent.
- Rinse thoroughly
- Air-dry
- Straps may be hand washed using a mild detergent soap and a soft brush with 86F (30 °C) warm water, then rinsed thoroughly and air-dried.

If brace is used in salt water or chlorinated water, rinse well and air dry.

Hinge maintenance

- Do NOT disassemble the hinge yourself
- Check regularly for foreign materials in hinge (i. e. dirt or grass)
- Clean dirt from hinge using water or compressed air
- Lubricate with silicone spray or dry graphite if needed

ACCESSORIES

The following accessories are available for your C180 brace at an additional charge. To order call your Ossur Customer Service Department.

- Patellar cup - Knee cap protection, especially for motocross, mountain biking or work that requires kneeling.
- Anti migration wrap

FIT ADJUSTMENTS

Widening or narrowing fit at knee joint:

The C180 brace can be adjusted for individual knee width. To widen or narrow the width at the knee, the 6 mm (1/4") padding located under each cover can be removed and replaced with thinner pads [3 mm ("1/8) or 1,5 mm ("1/16)] or you can install a shim behind the 6 mm (1/4") pad to make the fit more snug. Any combination of pads can be used to achieve a more comfortable fit.

To accommodate subtle variances in the anterior and posterior depth of the patient's leg, the anterior tibial strap (ATS) (**Figure 6**) should be loosened or tightened as needed so that the center of the hinges is slightly posterior of the midline. In conjunction, adjust the lower strap and the bottom strap. If the brace is too far anterior, loosen the tibial alignment strap and tighten the lower and bottom straps. If the frame is too far posterior, loosen the lower and bottom straps AND tighten the tibial control strap. Accurate depth setting will help to ensure optimal brace suspension

CHANGING EXTENSION STOPS

Extension stops are available for varying degrees of extension control.

1. Remove the cover and pad from the condylar area.
2. Using a short Philips head screwdriver, remove the screw in the center of the hinge plate (**Figure 7**).
3. Locate the extension stop positioned in the hinge between the upper and lower arms of the brace. With brace slightly bent, push the extension stop from the back of the brace through to the front. This should cause the stop to pop out of the hinge (**Figure 8**).
4. From the bag of extension stops included with the brace, choose the stop which corresponds to the desired degree of extension. Feed the

hole end of the stop through the hinge from the front to the back of the brace, with the curved side facing down and the flat edge facing up. Make sure that the sharp extrusions on the flat end of the extension stop are pointed toward the distal portion (bottom half) of the brace (**Figure 8**). The flat end of the stop should be visible from the front of the brace.

5. Align the hole in the center of the hinge plate with the hole in the extension stop. Replace and tighten the screw through the hinge plate and extension stop. Replace the pad and cover. This procedure must be completed on both the medial and lateral hinges.
 6. Choose the appropriate extension stop (Blue = 0°, Black = 0-10°, Dark Gray = 10-20°, Medium Gray = 20-30°, Light Gray = 30-40°)
- These numbers are approximate. Actual extension stop will vary from person to person depending on anatomical differences. Appropriate extension stop size is normally determined by health care provider.

LIMITED WARRANTY

This warranty covers all defects in materials and workmanship of your new C180 Knee Brace. This Warranty covers the rigid carbon fiber brace frame and Accutrac™ hinges for 2 years. The warranty covers all other materials, including straps, sleeves, liners and pads for six months from the date of original purchase.

Ossur will repair or replace (at its option) any defective part covered under warranty at no charge. Repairs necessitated by accident, negligence, misuse or abuse by the purchaser are not covered. Also, consequential and incidental damages are not recoverable under this warranty. These limitations may not apply as some states do not allow the exclusion or limitation of incidental or consequential damages. Only the original purchaser is entitled to claim service under this warranty. To obtain service under this warranty call our Customer Service Department.

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