Instructions for Use

AFO LEAF SPRING

EN		Instructions for Use	3
DE		Gebrauchsanweisung	4
FR		Notice d'utilisation	6
ES		Instrucciones para el uso	7
IT		Istruzioni per l'uso	8
NO		Bruksanvisning	9
DA		Brugsanvisning	10
SV		Bruksanvisning	11
FI		Käyttöohjeet	12
NL		Gebruiksaanwijzing	13
PT		Instruções de Utilização	15
PL		Instrukcja użytkowania	16
JA		取扱説明書	18
ZH		中文说明书	19

ENGLISH

INDICATIONS

- Drop foot secondary to CVA
- Mild drop foot secondary to other neurological pathologies

CONTRA -INDICATIONS NDICATIONS :

- Severe Drop foot
- Ankle-foot deformities
- M-L instability
- Long term bracing need
- Spasticity

WARNING

1. The AFO Leaf Spring is a supportive brace only and is not intended or guaranteed to prevent injury. Össur accepts no responsibility for any injury which might occur while using this product.
2. With the use of this type of device, it is possible to receive various types of skin irritation and or pressure points. If you experience any unusual discomfort while using this product, consult your health care provider immediately.
3. Patients with foot neuropathy should not use this product.

AFO LEAF SPRING SIZE SELECTION

Part	Product Name	Side	Height
O-110 127	Össur AFO Leaf Spring	R	10¾" (27 cm)
O-110 227	Össur AFO Leaf Spring	L	10¾" (27 cm)
O-110 130	Össur AFO Leaf Spring	R	12" (30 cm)
O-110 230	Össur AFO Leaf Spring	L	12" (30 cm)

FITTING THE AFO TO THE SHOE

- If present, remove the insole from the shoe.
- Adjust the shape of the footplate by grinding off excessive material.
- Make sure the plate covers the entire surface of the shoe sole to prevent displacement in the shoe.
- Ensure stability of the footplate on the shoe sole throughout the entire length of the shoe.

CUSTOMIZING THE AFO

- Insert the AFO into the shoe and have the patient don the AFO. If desired, temporarily attach the padding material.
- Check for excessive pressure points and, if necessary, remove the AFO from the patients leg and apply minimal heat (< 90°) to reshape the AFO to relieve these points.

FINISHING THE AFO

- If desired, permanently attach the padding to the AFO.
- Attach hook and loop straps using Speed Rivets.
- The AFO is now ready for use.

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