ENGLISH

CONTENT

- Fabrication Ring (**fig. 6**).
- Funnel (**fig. 2**).
- Ratchet Lock w/o adapter (**fig. 3**).
- Release Unit (**fig. 4**).
- Attachment Pin ratchet (L-792000) (**fig. 1**).
- Silicone Seal (**fig. 7**).

Items 6 and 7 are used when the socket is fabricated.

Use with L-780000 Fabrication Tooling (reusable).

Tools required

- 8mm (5/16) socket wrench.

INSTALLATION

- Place Ratchet Lock (3) into bottom of socket with release unit hole incorrect position.
- Shorten Release Unit (4) to required length and screw it firmly and completely into hole on socket and Ratchet Lock (3).
- OPTIONAL:** L-792016 Release Key (5) makes installation and removal of Release Unit (4) easier and is ideal when using prosthesis with exoskeletal design.
- Screw Funnel (2) into threads above Ratchet Lock (3) and torque to 8Nm (6ft.lb) using Cross Driver (8) (comes with Fabrication Tooling) and 8mm torque wrench.

ATTACHMENT PIN

Apply Loctite 410 or 411 onto threads on Attachment Pin (1) and screw it into Locking Liner. Torque Attachment Pin (1) to 4Nm (3ft.lb) with 6mm wrench

NOTE: Use Attachment Pins for Icelock 700 series only.

Icelock 700 Series

Important: For best results, ensure that the pin is fully engaged. If temporary swelling prevents full engagement, a minimum of four teeth must enter the lock for safety reasons.

Limited one-year warranty.

Distal/weight bearing adapter determines weight limit.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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