ENGLISH

ICELOCK® VALVE 641

L-641000

CONTENT

L-692022 Fabrication Ring w/ o-ring & silicone ring (**Figure 2**).

L-692116 Expulsion valve, silicone, (**Figure 1**).

L-693112 Silicone Seal (**Figure 3**).

L-693125 Replacement o-ring.

Items 2,3 & Replacement o-ring are used when the socket is fabricated.

Use with L-680000 Fabrication Tooling (reusable).

TOOLS REQUIRED

- 8mm (5/16) socket wrench.

INSTALLATION

- Following socket manufacture, replace existing o-ring with replacement o-ring to provide uncontaminated seal between Fabrication ring and Attachment.
- Screw Bottom Attachment w/ funnel and pre-inserted silicone Expulsion Valve (**Figure 1**) into bottom of socket.
- Torque to 8Nm (6ft.lb) using Cross Driver (**Figure 4**) (comes with Fabrication Tooling) and a 8mm torque wrench.
- Don socket and seal brim with an ICEFLEX suction suspension sleeve.

IMPORTANT

To prevent leakage, fasten the socket in a vice and torque the valve to 8 Nm (6ft.lb.) using the Cross Driver and 8mm (5/16) torque wrench.

CLEANING

It is recommended that Expulsion Valve (**Figure 1**) be cleaned at least every six weeks to ensure proper functioning. Clean with air and / or water & soap.

Distal/weight bearing component determines weight limit.

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

OTHER LOCKING COMPONENTS AVAILABLE FOR 600 SERIES

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