Instructions for use

## ICELOCK® 621 Adaptation Kit



Code number: L-621100

*Life Without Limitations®*

.....	3
ⒺⓃ Instructions for use.....	4
ⒹⒺ Gebrauchsanweisung.....	5
ⒻⓇ Guide de fabrication.....	6
ⒺⓈ Instrucciones para el uso.....	7
ⒻⓉ Istruzioni per l'uso.....	8
Ⓓⓐ Brugsanvisning.....	9
ⒻⓈ Bruksanvisning.....	10
ⒻⓃ Gebruiksaanwijzing.....	11
ⒻⓉ Instruções de Utilização.....	12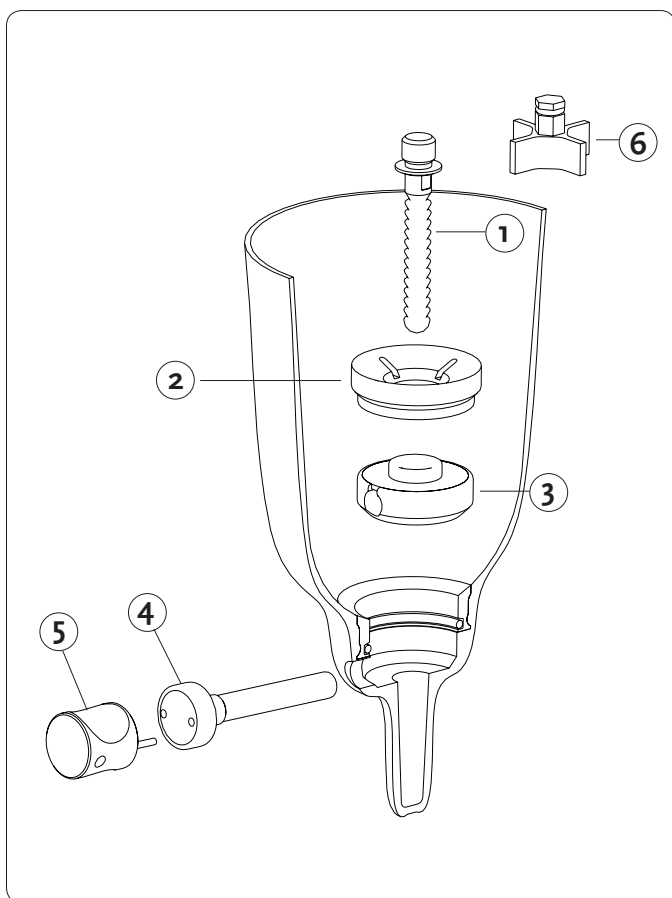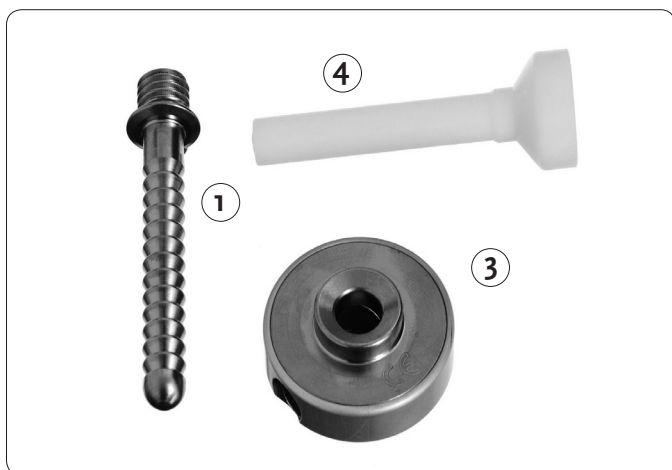

# ENGLISH

---

## CONTENT

- L-692020 Ratchet Lock w/o adapter (figure.3)
- L-692027 Release Unit, long (fig. 4)
- L-192000 Attachment Pin ratchet, standard (fig. 1)

## TOOLS REQUIRED

8mm (5/16) socket wrench

## INSTALLATION

- Place Ratchet Lock (3) into bottom of socket with release unit hole in correct position
- Shorten Release Unit (4) to required length and screw it firmly and completely into hole on socket and Ratchet Lock (3)  
**OPTIONAL:** L-692016 Release Key (5) makes installation and removal of Release Unit (4) easier and is ideal when using prosthesis with exoskeletal design
- Screw Funnel (2) into threads above Ratchet Lock (3) and torque to 8Nm (6ft.lb) using Cross Driver (6) (comes with Fabrication Tooling) and 8mm torque wrench

## ATTACHMENT PIN

Apply Loctite 410 or 411 onto threads on Attachment Pin (1) and screw it into Locking Liner. Torque Attachment Pin (1) to 4Nm (3ft.lb) with 7mm wrench.

If needed, L-192003 Attachment Pin ratchet short and L-192005 Attachment Pin ratchet long are available

**NOTE:** Use Attachment Pins for ICELOCK 600/100 series only.

### Limited one-year warranty

Rated up to high activity level (K3)

Distal/weight bearing adapter determines weight limit

Other locking components available for 600 series

L-631100 ICELOCK lanyard 631

L-641100 ICELOCK valve 641

L-651100 ICELOCK smooth lock 651

All locks are fully interchangeable without necessitating fabrication of a new socket.

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**EN - Caution:** Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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### **Össur Iberia S.A.**