## ENGLISH

### L-331002

- Attach the Delrin Guide **(1)** to the cast with the four nails **(2)** provided.
- Starting at the edge of the Guide, drill a small hole with a wire drill through the full length of the plaster cast to its base. This assures maximum vacuum at the distal end.
- Secure the Bottom Attachment **(3)** to the Guide with the Philip head screw **(4)**. Make sure that the exit hole, with the dummy **(5)** for the Lanyard Cord, is correctly positioned to ensure that the Lanyard operates with minimum friction.
- Vacuum-form the thermoplastic, ensuring that the plastic shapes accurately to the Bottom Attachment.
- After manufacturing the outer socket, trim around the screw and the exit hole and remove the screw and the dummy.
- Assemble the socket and drill a 5mm hole through the outer socket - opposite and slightly higher than the exit hole.
- Thread the Lanyard Cord **(6)** through the Bottom Attachment and out through the exit hole.
- Find the correct positioning for the Cleat **(7)**; drill two 3mm (1/8") holes and rivet it to the socket.
- Trim the Cord Cover **(8)** to the desired length, insert it into the exit hole and glue it to the socket in the correct location, using Loctite 425.

### Cleat Positioning (Figure 3)

**(10)** Released  
**(11)** Locked

**Important:** Tie a knot **(9)** in the Lanyard Cord to stop it from inadvertently being pulled into the channel.

**Note:** The Pull Handle L-392016 (not shown) can be used for getting a more secure grip on the Lanyard Cord when the socket is donned.

- It is recommended that the Lanyard Cord be regularly checked for wear
- Rated up to high activity level (K3)

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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**Össur Deutschland GmbH**