English

French

## ENGLISH

### CONTENT

Lanyard Cord with connecting screw **(1)**

Funnel **(2)**.

Bottom Attachment, lanyard **(3)**.

Elbow, lanyard **(4)**.

Cord Cover 40cm **(5)**.

Cleat with cover and rivets **(6)**.

Pull Handle, lanyard **(7)**.

Fabrication Ring w/ o-ring **(8)**.

Silicone Seal **(9)**.

Cross Driver **(10)**.

*Items 8 and 9 are used when the socket is fabricated.*

**Use with L-680000 Fabrication Tooling (reusable).**

### Tools required

- 8mm (5/16) socket wrench.

### INSTALLATION

- Place Bottom Attachment **(3)** into bottom of socket with hole for Lanyard Cord **(1)** in correct position.
- Shorten Elbow **(4)** to required length and insert it into hole.
- Screw Funnel **(2)** into threads above Bottom Attachment **(3)** and torque to 8Nm (6ft.lb) using Cross Driver **(10)** (comes with Fabrication Tooling) and 8mm torque wrench.
- Find correct positioning for Cleat **(6)**, drill two 3mm (1/8”) holes, and rivet Cleat **(6)** to socket.
- Trim Cord Cover **(5)** to desired length. Bond it into Elbow **(4)** and to hard socket in alignment with *“tunnel”* in Cleat **(6)**.
- Thread Lanyard Cord **(1)** all the way through Bottom Attachment **(3)**, Elbow (4) and Cord Cover **(5)**.
- Screw connecting screw on Lanyard Cord **(1)** into distal end of Locking Liner, don socket and secure Cord in Cleat **(6)**.

**Important:** Tie knot **(11)** in Lanyard Cord **(1)** to stop it from inadvertently being pulled into channel.

**Note:** Pull Handle **(7)** can be used for a more secure grip on Lanyard Cord **(1)** when socket is donned.

1 year limited warranty.

It is recommended that Lanyard Cord **(1)** be regularly checked for wear. Rated up to high activity level (K3). Distal/weight bearing adapter determines weight limit.

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

**Other locking components available for 600 series:**
L-621000 Icelock ratchet 621.
L-641000 Icelock valve 641.
L-651000 Icelock smooth lock 651.

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**EN – Ossur:** Ossur products and components are designed according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Ossur products are used with other recommended Ossur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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©Copyright Ossur 2017 IFU 0035 IFU 1226\_001 Rev. 5