ENGLISH

ICELOCK 400 SERIES

ASSEMBLY KIT PART NUMBER P-20192

For the ALP system with laminated socket P-20192

CONTENT

P-20177	Step-less Distal Attachment
P-20170	Locking Pin
P-20162	Funnel
P-20142	Coupling Plate

To produce a socket with P-20192 you also need

P-20109	Fixture (reusable)
P-20152	Dummy (reusable)

TOOLS REQUIRED

23mm	Drill Bit
3mm	Drill Bit
	Cross Head Screw Driver

SOCKET FABRICATION

- Flatten distal end of plaster cast and attach Fixture (P-20109) **(1)** with two small nails. Transition from cast to Fixture should be smooth. Ensure Fixture is mounted in alignment with residual limb.
- Fabricate a soft liner of 3mm PeLite or similar material **(2)**. Leave the distal bolt end open. Mount the bolt **(3)**.
- Apply Dummy (P-20152) and Coupling Plate (P-20142) on top of Fixture **(4)**. In the event of a gap between the Dummy and soft liner, fill with PeLite or similar **(5)**.
- Insulate with PVA-bag and seal at the top of the Coupling Plate.
- Reinforce the PVA-bag with polythene tape around the Coupling Plate.
- Apply the ALP-adaptor on top of the Coupling Plate with correct insert and laminate as usual **(6)**.
- Remove socket from the cast and also the Fixture, Dummy and the Coupling Plate**(7)**. Drill a 23mm hole for the push button.
- Attach the Coupling Plate to the Attachment (P-20177) with enclosed 6 screws. Put the Attachment and Coupling Plate into position within the socket **(8)**.
- Drill 4x 3mm holes through the socket wall into the flange of the Coupling Plate. Secure the Coupling Plate to the socket with enclosed 4 screws **(9)**.
- Trim the push button to desired length and apply to the attachment **(10)**.
- Apply the Funnel (P-20162) to the Attachment from the inside of the socket **(11)**.

Note:

- Do always check that the threaded adaptor for the locking pin is tightened before the prosthesis is applied.
- The product has to be checked at least once a year by a prosthetist.
- The locking pin should be exchanged at least once a year.
- In case of replacements, only use original components.
- Maximum weight of the prosthesis: 10 kg.

Cleaning The Pin Step-less Distal Attachment:

Unscrew the Push Button. Blow with compressed air alternately between the Locking Pin hole and the Push Button hole. **NOTE!** Do not use lubricant or cleaning material. Only use air.

One Year Warranty Rated for all Impact levels

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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©Copyright Össur 2017 IFU 0184 1311_001 Rev. 3