Instructions for use

Code number:

- Ⓔ EN Instructions
- ⒹⒺ DE Einbauanleitung
- ⒻⒺ FR Guide de fabrication
- ⒺⒺ ES Instrucciones de fabricación
- ⒻⒺ IT Istruzioni di Fabbricazione
- ⒹⒺ DA Brugsanvisning
- ⒺⒺ SV Tilverkningsanvisning
- ⒻⒺ PO Instruções de Utilização
- ⒹⒺ DU Gebruiksaanwijzing voor de vervaardiging

TALUX® Achilles Strap

Product codes: TLX50001
TLX50002
TLX50003
TLX50004

Life Without Limitations

ENGLISH – *Caution:* OSSUR products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when OSSUR products and components are used with other recommended OSSUR or authorized components.

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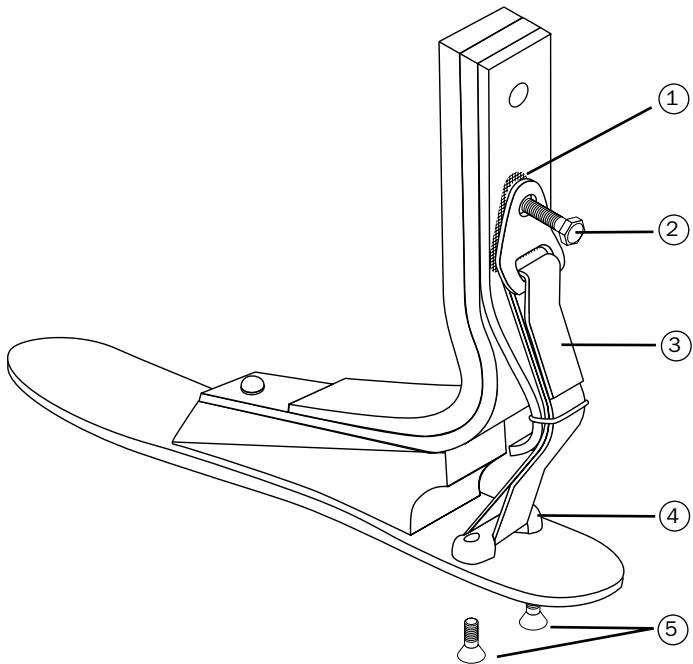
INSTALLATION OF STRAP ASSEMBLY

- I. Remove existing Achilles strap from foot module.
 - i. Remove two M5 cap screws from the anchor with a #2 Phillips driver.
 - ii. Remove the proximal Connector Mounting Bolt with an appropriate size socket wrench.

- II. Install new Achilles strap to foot module.
 - i. Slide the anchor through the distal loop of the Achilles strap assembly. Orient strap as shown and position anchor over holes in footplate.
 - ii. Apply Loctite 243 to the threads of the M5 screws and torque to the following specification:

Cat.	Bolt Size	Torque Specification
1-8	M5	4.1Nm (3.0 ft-lbs)

- iii. Insert friction pad between foot module and hanger. Ensure that hanger is properly aligned with slot parallel to the ground. Misalignment of the hanger may result in premature wear.
- iv. Apply Loctite 410 to the threads of the proximal Connector Mounting Bolt and torque to the following specifications:

Cat.	Bolt Size	Torque Specification
1-6	5/16"	22Nm (16ft-lbs)
7-8	3/8"	34Nm (25ft-lbs)
1-6*	M8	22Nm (16ft-lbs)
7-8*	M10	34Nm (25ft-lbs)

*Torque specifications apply to bolts used with integrated male pyramid connector.

NOTE: Slight compression of the Tarsal Core may be necessary to create slack in the strap assembly and ease insertion of the Connector Mounting Bolt into the foot module.

Caution: Don't use the alignment wedges with the male pyramid adapter on the TALUX®

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