ENGLISH

A-135100

- Material: Titanium
- Weight Limit: 166kg
- Weight: 58g

Exclusive use for Transtibial laminated sockets. Do not bend adapter.

INSTRUCTIONS FOR INTEGRATION INTO LAMINATED SOCKET.

- Isolate cast with PVA bag.
- Apply Double layer of Perlon Stockinet.
- Apply single 50mm carbon tapes crossed over distal end of cast, in AP/ML direction.
- Position Lamination cap on Pyramid Socket Adapter and secure by tightening central bolt using a 4mm Hex Key.
- Position Socket Adapter and Lamination Cap on model in correct alignment and rotation.
- Apply Lay up material and combinations of material appropriate to the individuals weight and impact/activity level.
- Layers of material may be trimmed to different lengths to provide a thinner lighter proximal socket.
- Lamination material should be tied into groove on Socket Adapter.
- Laminate with 80:20 acrylic lamination resin.

The Lamination Cap should be removed following lamination by unscrewing the central bolt using a 4mm Hex key.

- 2 year limited warranty.**

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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www.ossur.com ©Copyright Össur 2017 IFU 0097 1165_001 Rev.6
Össur Deutschland GmbH