Instructions for Use





---

## ICEROSS<sup>®</sup> UPPER-X

Part number: I-813112, I-813114, I-813116,  
I-813118, I-813120, I-813122

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	10
ES	Instrucciones para el uso	13
IT	Istruzioni per l'uso	16
DA	Brugsanvisning	19
SV	Bruksanvisning	22
NL	Gebruiksaanwijzing	25
PT	Instruções de Utilização	28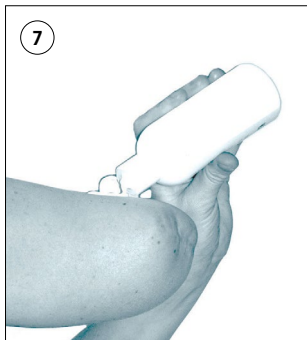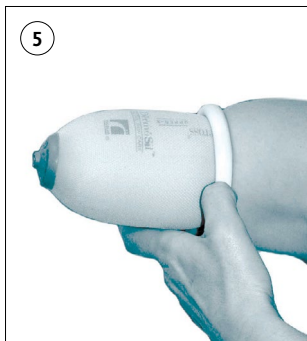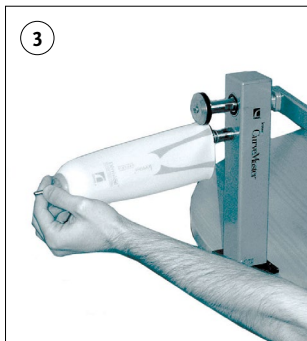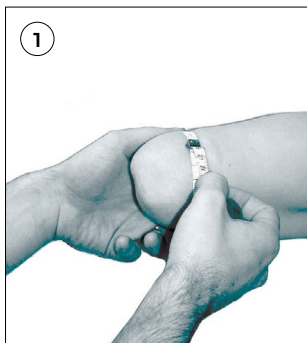

## INTRODUCTION

Thank you for choosing a high quality silicone liner from Össur. We can assure you that you have made the right choice. Please carefully review and follow these instructions before fitting a patient with the liner or accessory products. Remember that informing the patient concerning proper handling and care of the liner, as set forth in these instructions, is important to achieve patient comfort and maximum product life.

## PRODUCT SELECTION AND FITTING

### ***Liner Sizing (Figure 1)***

To determine the correct Iceross size:

- A. Measure the circumference of the residual limb 4cm from the distal end.
- B. Choose the Iceross size which corresponds to the measurement or which is immediately below the indicated measurement.
- C. Never choose a liner size which exceeds the measurement or round up.

**EXAMPLE:** If the residual limb measures 19cm at the indicated location, choose Iceross size 18 NOT size 20.

Selecting the correct size is very important. If the liner is too tight, this may cause pistoning and / or numbness and blistering. If the liner is too loose, increased perspiration and / or movement of the limb inside the liner may occur. Perspiration and movement can lead to blisters and rashes. If any of the above symptoms or any other indication of improper fit are noted, the user should be instructed to contact his / her physician or prosthetist immediately.

### ***Distal Attachments (Figure 2)***

Caution must be taken in fitting the distal attachment to the locking liner. Proper distal attachment is required to ensure user safety. Össur has introduced a smaller thread for upper extremity use and recommends the use of a lock from the Icelock 700 series together with Iceross Upper-X. Most available lower extremity locks will not be compatible. Always apply Loctite 410/411 or equivalent compound to the threads of the Össur attachment pins (used with Icelock) and torque to 4 Nm (3 ft./lbs.). If excessive movement or product wear is detected between the liner and the pin or between the pin and the distal attachment at any time, the patient should be instructed to immediately discontinue use of the device and consult his / her physician or prosthetist.

### ***Trimming an Iceross Liner (Figure 3)***

Iceross liners may be trimmed for user preference and to increase range of motion. The Iceross CurveMaster is specifically designed for this function and produces a smooth, rounded edge, which will minimize the risk of skin irritation and / or tears in the liner. Do not cut the below the proximal trim lines of the socket. Excessive trimming may reduce the natural suction between the liner and limb and / or compromise suspension.

### ***Donning an Iceross Liner (Figure 4 - 6)***

The Iceross Upper-X is delivered rolled up into a doughnut shape. It

should always be donned by unrolling the doughnut onto the residual limb. Exposing as much of the distal end of the liner as possible, position it against the residual limb and with light compression unroll the Iceross upward onto the limb. Check that no air pockets are present.

**NOTE:** Be very careful that the inside of the liner is clean, dry and free from any foreign objects that can cause skin irritation.

Roll the liner all the way up the limb, taking care not to damage it with your fingernails. Do not tug or pull.

### **Myo-electric prostheses**

Iceross Upper-X can successfully be used with myoelectric prostheses. Where the length of the residual limb allows, the Iceross can be cut below the level of the electrodes. If a longer liner is required, it is possible to use a sharp hole punch of the appropriate size to cut holes in the liner to facilitate direct skin contact for the electrodes. If holes are made, the cover on the edges should be smeared with silicone adhesive to avoid delamination.

## **CARE**

### ***Liner Care (Figure 7)***

Proper product care is important. Following use, the liner needs to be washed inside and out every day. Remove the liner, turn the liner inside out and wash it with pH balanced, 100% fragrance- and dye-free soap. The liner can also be machine washed (40 -60°C) with a mild detergent. Fabric softeners or bleaches or other products or cleaning solutions may damage the liner and should NOT be used. After cleaning, the patient should rinse the liner thoroughly with water and wipe both sides dry with a lint-free cloth.

An Iceross liner can be used directly after washing. It does not need to dry overnight. Always return the liner to its normal position with the distal attachment facing out as soon as possible after cleaning.

**NOTE:** Care must be taken by both the prosthetist and the patient not to expose a liner to glass or carbon fibres or other foreign particles. Such substances can become embedded in the silicone, causing aggravation to the skin. Washing alone may not be sufficient to eliminate the problem and the patient should be instructed to return the liner to the prosthetist if the liner is inadvertently exposed to any foreign substances or chemicals.

DermoSil™ and DermoGel™ silicone contain Aloe Vera and Vaseline. They act as a natural moisturizers and Aloe Vera is known for its soothing effects on the skin. Initially, the inner surface of the liner should be slightly moist to the touch. For these reasons, the liner should not be left inside out for an extended period or exposed to excessive heat or sunlight.

### ***Skin Care***

Daily cleaning of the residual limb is also essential. We recommend use of a mild liquid, pH balanced, 100% fragrance and dye-free soap. If dry skin is noted, apply a pH balanced, 100% fragrance- and dye-free lotion to nourish and soften the skin. Do NOT apply lotion to the residual limb immediately prior to donning the liner.

**NOTE:** Many common household or bath products, including soap, deodorants, perfumes, aerosol or alcohol sprays or abrasive cleaners may cause or contribute to skin irritation and the patients should be so advised.

#### **PRODUCT WARRANTY**

Össur offers a written warranty for the indicated period from the invoice date. Said warranty shall be in lieu of all others, whether express or implied by law.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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